## If you are being treated with Novolizer®

## Novolizer® - easy to use correctly

Push down the dispenser button to prepare a dose. When the indicator turns green, the dose is in place and the inhaler is ready to use. After a correct inhalation the display changes from green to red. If the colour does not change, inhale again.



1. Take off the cap




2. Press the dispenser button



3. Inhale

### Novolizer® - you know when you've done it right

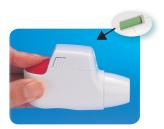



# **Double click**Confirms the dose is ready



**Green**The inhaler is ready to use





#### Click Confirms correct inhalation



#### Red Confirms correct inhalation



Taste Confirms correct inhalation



Scan the QR code with your mobile phone to see instructional video at www.novolizer.se

## Novolizer® – a clear dose counter

Novolizer® has a dose counter that shows every tenth dose. When the dose counter shows 0, it's time to change the cassette.




#### Visit our websites:

- www.novolizer.se
- www.astmalinjen.se
- www.kolguiden.se



## Refilling the inhaler with a new cassette


- Lightly press together the ribbed surface on the sides of the cover, slide the cover forward and lift it off.
- 2. Remove the protective aluminium foil from the cassette holder and take out the cassette.

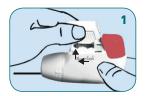

Place the cassette in the inhaler with the dose counter facing forward.





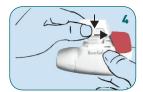

#### Novolizer® – the inhaler that makes a difference

Novolizer® is used in the treatment of asthma and COPD (chronic obstructive pulmonary disease). Novolizer® comes in a series:



**Novopulmon® Novolizer®** (budesonide) Inhalation steroid

200 μg, 200 doses • 400 μg, 200 doses



Formatris® Novolizer® (formoterol) Long-acting bronchodilators

6 μg, 60 doses • 12 μg, 60 doses



**Ventilastin® Novolizer®** (salbutamol) Short-acting bronchodilators

100 μg, 200 doses

Novolizer® lasts for up to 2000 inhalations or for one year. This means you can refill the inhaler yourself, which reduces the cost and the environmental impact.

### My treatment plan

Novopulmon® Novolizer® - inhalation steroid

Strength:  $\square$  200  $\mu$ g  $\square$  400  $\mu$ g

Number of inhalations/day: ...... Start date: .....

Ventilastin® Novolizer® — short-acting bronchodilators

Strength: ☐ 100 µg

Number of inhalations/day: ...... Start date: .....

Formatris® Novolizer® - long-acting bronchodilators

Strength: □ 6 μg □ 12 μg

Number of inhalations/day: ...... Start date: .....

